

DATA SHEET

MEDICAL FACE MASK

Used in the general medical environment, public health places in the general health care

TESTING ACCORDING TO EN 14683: 2019 Type II

Bacterial filtration efficiency (BFE), (%)	≥ 98
Differential pressure (Pa/cm ²)	< 40
Splash resistance pressure (kPa)	Not required
Microbial cleanliness (cfu/g)	≤ 30

INSTRUCTIONS FOR FITTING:

1. Take out the mask, white face inward, blue face outward.
2. Align the face mask with the mouth and nose, with nose clip upward, and press the nose clip gently to make it fit with nose;
3. Hang the ear belt on the ear (or tie it to the head), and adjust the ear belt/tie to a comfortable state;
4. Adjust the wearing of the face mask to cover only the lower jaw of the nose, so as to obtain better protective effect.